INSTRUCTIONS FOR USE **INSTRUCTIONS D'UTILISATION**

FarrowWrap® / Farrow / FarrowHybrid®
assortment / gamme

Adjustable compression devices
Non-compressive liners
Medical compression garments
Accessories

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MATERIAL COMPOSITION / COMPOSITION

| Product / Produit | | % Polyamide (Nylon) | % Elastane / Élasthanne (Spandex) | % Cotton / Coton | % Polyester | % Neoprene / Néoprène | % Polyurethane / Polyuréthane |
|---|----------------|---------------------|-----------------------------------|------------------|-------------|-----------------------|-------------------------------|
| Adjustable compression devices / Dispositifs de compression ajustables | | | | | | | |
| FarrowWrap® LITE | Fabric / Tissu | 82 | 18 | | | | |
| | Foam / Mousse | | | | | | 100 |
| FarrowWrap® LITE Handpiece / Dispositif pour la main | Fabric / Tissu | 89 | 11 | | | | |
| | Foam / Mousse | | | | | | 100 |
| FarrowWrap® STRONG | Fabric / Tissu | 82 | 18 | | | | |
| | Foam / Mousse | | | | | | 100 |
| FarrowWrap® CLASSIC | Fabric / Tissu | 32 | 5 | 47 | 16 | | |
| | Foam / Mousse | | | | | | 100 |
| FarrowWrap® BASIC | Fabric / Tissu | 89 | 11 | | | | |
| | Foam / Mousse | | | | | | 100 |
| FarrowWrap® 4000 | Fabric / Tissu | 89 | 11 | | | | |
| | Foam / Mousse | | | | | | 100 |
| Non-compressive liners / Doublures non compressives | | | | | | | |
| Farrow Liner / Doublure | | 96 | 4 | | | | |
| Farrow Terry Cloth liner / Doublure | | | 15 | 85 | | | |
| Medical compression garments / Vêtements de compression médicale | | | | | | | |
| Farrow Glove / Gant | | 59 | 41 | | | | |
| Farrow Toe Cap / Couvre-orteils | | 59 | 41 | | | | |
| FarrowHybrid® ADI | | 83 | 17 | | | | |
| Accessories / Accessoires | | | | | | | |
| Farrow GarmentGrip Roll / Rouleau de néoprène | | 48 | 5 | | | 47 | |

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JOBST® FARROWWRAP® BENEFITS / PRESTATIONS

- EN** FarrowWrap® / Farrow / FarrowHybrid® Features
- DE** Eigenschaften von FarrowWrap® / Farrow / FarrowHybrid®
- FR** Caractéristiques des dispositifs FarrowWrap® / Farrow / FarrowHybrid®
- ES** Características de FarrowWrap® / Farrow / FarrowHybrid®
- PT** Características da seleção de produtos FarrowWrap® / Farrow / FarrowHybrid®
- IT** Caratteristiche di FarrowWrap® / Farrow / FarrowHybrid®
- NL** FarrowWrap® / Farrow / FarrowHybrid® Kenmerken
- SV** FarrowWrap® / Farrow / FarrowHybrid® Egenskaper
- NO** FarrowWrap® / Farrow / FarrowHybrid®- egenskaper
- DA** FarrowWrap® / Farrow / FarrowHybrid®: Funktioner
- FI** FarrowWrap® / Farrow / FarrowHybrid® tuotteiden ominaisuudet
- IS** FarrowWrap® / Farrow / FarrowHybrid® tuotteiden ominaisuudet
- EE** FarrowWrap® / Farrow / FarrowHybrid® omadused
- LT** FarrowWrap® / Farrow / FarrowHybrid® Savybės
- LV** FarrowWrap® / Farrow / FarrowHybrid® funkcijas
- PL** Cechy produktów FarrowWrap® / Farrow / FarrowHybrid®
- HU** FarrowWrap® / Farrow / FarrowHybrid® jellemzői
- RO** Caracteristicile produselor FarrowWrap® / Farrow / FarrowHybrid®
- CZ** Vlastnosti soupravy FarrowWrap® / Farrow / FarrowHybrid®
- SL** Funkcije izdelkov FarrowWrap® / Farrow / FarrowHybrid®
- RU** Свойства товаров FarrowWrap® / Farrow / FarrowHybrid®
- TR** FarrowWrap® / Farrow / FarrowHybrid® Özellikleri
- AR** / Farrow / FarrowWrap® ميزات FarrowHybrid®
- HE** תכונות ההתקן FarrowWrap® / Farrow / FarrowHybrid® של

JOBST® FARROWWRAP® BENEFITS/PRESTATIONS

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INSTRUCTIONS FOR USE

Dear Customer,
Thank you for choosing
FarrowWrap®/Farrow/FarrowHybrid®.

This assortment includes adjustable compression devices for upper and lower extremities, non-compressive liners, medical compression garments and related accessories.

1. Intended use and indications

Please follow your healthcare provider's instructions.

FarrowWrap®/Farrow/FarrowHybrid® products are designed to exert physical, external therapeutic compression to help specific venous and lymphatic conditions and swelling of other origins.

They have a specific compression level which must be determined by your physician for the management of the following indications:

Lower limb

FarrowWrap® LITE, FarrowHybrid® ADI, Farrow Toe Cap

Mild to moderate swelling from venous, lymphatic and other swelling disorders; mild to moderate post thrombotic syndrome (PTS); chronic venous disease (CVD, C1-C5).

FarrowWrap® STRONG, FarrowWrap® CLASSIC, FarrowWrap® BASIC, FarrowWrap® 4000

Moderate to severe swelling from venous, lymphatic and other swelling disorders; chronic venous disease (CVD, C3-C6); venous leg ulcer; mild to moderate post thrombotic syndrome (PTS).

Upper limb

FarrowWrap® LITE, Farrow Glove

Mild to moderate swelling from lymphatic and other swelling disorders.

FarrowWrap® STRONG, FarrowWrap® CLASSIC

Moderate to severe swelling from lymphatic and other swelling disorders.

Non-compressive liners are stockings/sleeves for use under FarrowWrap® adjustable compression devices to help protect the skin.

Accessories comprise Farrow GarmentGrip, used with FarrowWrap® adjustable compression devices to help maintain hold and Farrow Foams, designed to evenly distribute the pressure and to add padding under FarrowWrap® adjustable compression devices. FarrowWrap® Strong/Lite TTF VELCRO® packs assists the medical functionality of the medical devices FarrowWrap® Strong and Lite adjustable compression devices in terms of their intended purpose.

FarrowWrap® Replacement assists the medical functionality of the medical device FarrowWrap® Gauntlet in terms of its intended purpose.

FarrowWrap® Short Stretch Band assists the medical functionality of the medical devices FarrowWrap® Strong and Classic adjustable compression devices in terms of their intended purpose.

Please contact your healthcare professional if you are unsure whether the product is appropriate for you.

2. Directions for use

Follow the product specific donning instructions provided in the back of the insert card.

Please also follow your healthcare provider's instructions.

Application videos, are available at:
www.jobst.com

To avoid pressure marks, smooth out any wrinkles, while donning.

To achieve the indicated compression (mmHg), FarrowWrap® garments should be stretched as follows:

LITE & STRONG

Fix the VELCRO® brand pieces applying a moderate stretch which resembles an approximate stretch of the garment by 15%.

CLASSIC & BASIC

Fix the VELCRO® brand pieces applying a gentle stretch which resembles an approximate stretch of the garment by 10%.

INSTRUCTIONS FOR USE

4000

Fix the VELCRO® brand pieces applying a stretch according to the values stated in the table below:

| XS | S | M | L |
|-----------|---|-----------|---|
| 15 - 20 % | | 20 - 30 % | |

PLEASE NOTE:

- Initial fitting should be done under the supervision of a health care professional, to determine comfortable compression level; based on patients need and tolerance.
- FarrowWrap® TTF must be trimmed by a medical professional prior to any donning.
- If it has not been trimmed, please contact your provider for further information.
- Following application, if the patient exhibits signs of skin reddening, strictures, pressure marks, pain, reduced color or numbness in the extremities, the compression wrap system should be removed.
- It is recommended to wear a FarrowWrap® brand liner under FarrowWrap® adjustable compression devices to help protect the skin.
- Slowly pull liner over toes and up the leg, slightly pull the end away from the toes so that they have room to wiggle and smooth out any wrinkles. Make sure liner is correctly sized and appropriately donned.
- The liner should be changed in accordance with daily hygiene and wound inspection needs.
- Consult your health care professional with regards to combining different FarrowWrap® products.
- FarrowWrap® kneepiece LITE should only be used in combination with FarrowWrap® thighpiece LITE.
- FarrowWrap® kneepiece STRONG should only be used in combination with FarrowWrap® thighpiece STRONG and CLASSIC.
- It should be considered combining the FarrowWrap® footpiece with a FarrowWrap® legpiece for the management of legedema.
- It may be necessary to loosen the VELCRO® brand pieces of the garment during night-time wear.
- For FarrowHybrid®, Farrow glove and Farrow Toe Cap it is recommended to consider removal when lying down.

- Please reach out to your health care professional for further advice.
- Jewelry in the affected area should be removed before donning.
- Excess liner can be folded down over the top of the compression garment.
- Garment may need to be tightened periodically to ensure correct fit.
- To prevent the VELCRO® brand pieces from sticking to other items, roll each band back onto itself and affix VELCRO® brand pieces.

3. Contraindications

FarrowWrap®/Farrow/FarrowHybrid® products are contraindicated (should NOT be worn) if you have one or more of the following medical conditions:

- Vein infection (septic phlebitis)
- Uncontrolled (decompensated) congestive heart failure
- Untreated and/or advancing skin infection (cellulitis) involving the area where the garment will be applied
- Phlegmasia coerulea dolens (large blood clot in the vein of the area treated)
- Patients unable to express pain or discomfort
- Circumference less than the indicated ranges (see sizing chart in the insert cards)
- Incompatibility to material

Compression 20 - 30 mmHg and 15 - 20 mmHg for Farrow Toe Cap/Farrow glove

- Severe arterial disease
- Moderate arterial disease in the presence of peripheral neuropathy

Compression 30 - 40 mmHg

- Moderate to severe arterial disease
- Mild arterial disease in the presence of peripheral neuropathy

Note: JOBST® Farrow products are designed to be used together and should not be used with non-JOBST® Farrow products.

4. Cautions

- Open wounds must be covered with an appropriate dressing before the liner and/or compression garment is put on.
- For lower limb garments, appropriate footwear must be worn when walking to help prevent falls.

Consult your physician if you have experienced or develop:

- Skin infection or weeping
- Mild to moderate arterial disease
- Loss of feeling in the area where the garment has been applied (peripheral neuropathy)
- History of blood clots
- Controlled CHF (congestive heart failure)
- Carpal Tunnel Syndrome (only for upper limb)
- Large bunions (only for Farrow Toe Cap, FarrowHybrid®)
- Immobility (only for Farrow Toe Cap)
- Conditions in which increased venous and lymphatic return is not desired

5. Size table

For standard sized garments (Ready-to-Wear & Trim-to-Fit): please consult the sizing chart in the insert card.

6. Directions for storage / Guarantee

Please protect your compression garments from direct exposure to sunlight, heat, and moisture.

We recommend storage at room temperature.

The wearing period is not more than 6 months for JOBST® Farrow products.

The manufacturing date/expiry date for storage is printed on the package label next to the corresponding symbol.

Made-to-Measure products are products intended for immediate use and no expiry date for storage is indicated on these.

7. Notice

Any serious incident that has occurred in relation to this device should be reported to BSN medical Inc. (BSN medical GmbH within the European Union) and the competent authority of your state.

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VELCRO® is registered trademark of Velcro BVBA./
Essity UK Ltd · Southfields Road · DUNSTABLE · Beds · LU6 3EJ
BSN-Jobst GmbH
BSN medical (Aust.) Pty Ltd