JOBST®

INSTRUCTIONS FOR USE

Medical compression stockings

JOBST® Confidence

- Ⓔ Medical compression stockings
- Ⓓ Medizinische Kompressionsstrümpfe
- Ⓕ Bas médicaux de compression
- Ⓔ Medias de compresión médica
- Ⓔ Medicinska kompressionsstrumpor
- Ⓔ Medisinske kompresjonsstrømper
- Ⓓ Medicinsk kompressionsstrømpe

- Ⓔ Medische compressiekousen
- Ⓓ Meias de compressão medicinais
- Ⓓ Medyczne pończosznice wyroby kompresyjne
- Ⓓ Kompressziós térdharisnya
- Ⓓ Calze compressive medicali
- Ⓓ Lääkinnälliset kompressiosukat
- Ⓓ Ciorapi compresivi medicinali

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FIBER CONTENT

Brand / CCL	Polyamide (PA)	Polypropylene (PP)	Elastane (EL)
JOBST Confidence 1	59%	26%	15%
JOBST Confidence 2	48%	32%	20%
JOBST Confidence 3	45%	32%	23%

Dear Customer,

Thank you for choosing JOBST®, the medical compression stockings that incorporates gradient compression for the management of venous and lymphatic diseases. JOBST® offers you a wide selection of products for these indications.

1. Application

Tips for Wearing

JOBST® Confidence Leg Wear:

To get the best medical result from your JOBST® Confidence garment, wear it regularly and consistently.

Ideally you should put on your compression stocking in the morning, immediately after getting up. You may experience some difficulty putting on the stocking in the beginning, but this will become easier after a number of days.

When applying the compression stockings, please do not pull on the edge of the stockings instead follow these recommendations below.

Turn the compression stocking inside out all the way to the heel. Pull apart the folded-in foot part using both hands and pull over the foot all the way to the heel.

Gradually pull the compression stocking over instep and heel.

Gradually pull the compression stocking upward by returning the stocking to a right side out position piece-by-piece.

Distribute the stocking evenly by massaging with both hands until the stocking fits and does not form creases. It is particularly helpful to wear rubber gloves for this step.

Special notes:

- Caution: long fingernails and jewelry may damage the stockings when putting them on. For easier application use rubber gloves or a donning aid.
- Do not bring the compression stockings into contact with greasy ointments, body lotions or similar substances unless recommended by JOBST®.
- Loose threads or loops should not be cut off, since this may cause holes or runs to form in the stockings.
- Intended to be used by a single patient only.

2. Care instructions

JOBST® compression stockings should, if at all possible, be washed daily, or at least every other day.

For this purpose, you may use a regular mild detergent. Do not use laundry additives such as fabric softener, optical brightener or stain remover. Rinse your compression stockings well.

Roll up the wet compression stockings in a towel after washing and press out excess moisture. Do not wring. Do not dry the compression stockings on a radiator or in direct sunlight.

Silicone band care:

When needed, hand wash silicone band to remove excess creams, lotions, hair and other foreign material.

3. Indications

Every JOBST® Confidence leg wear product has a specific compression level designed to help specific venous and lymphatic conditions. The compression level is printed on the package label. The required compression class must be determined by your physician. Medical compression garments are used for the management of the following indications:

Compression Class 1 (CCL 1)

- Tired, heavy, aching legs
- Minor varices without edema
- Mild swelling of the feet, ankles and legs
- Onset of pregnancy-related varices
- Primary or secondary lymphedema in International Society of Lymphology (ISL) stages I-II with no shape distortion

Compression Class 2 (CCL 2)

- Tired, heavy, aching legs
- Moderate to severe varices with and without edema
- Moderate to severe pregnancy-related varices with and without edema
- Moderate edema
- Chronic venous insufficiency (CVI) C3-C5 according to CEAP: venous edema, skin alterations and healed ulcer cruris
- Helps to prevent the recurrence of venous ulcers
- For adjunct management of superficial thrombophlebitis/SVT
- Post vein surgery
- Post sclerotherapy
- Post-traumatic and post-operative edema
- Primary or secondary lymphedema in ISL stage II with no shape distortions
- Phlebolympheidema
- Lipedema

Compression Class 3 (CCL 3)

- Severe varices with or without edema
- Severe edema
- Chronic venous insufficiency (CVI) C3-C6 according to CEAP: venous edema, skin alterations, healed venous leg ulcer and leg ulcer
- To prevent recurrence of venous ulcers
- Post-thrombotic syndrome (PTS)
- For adjunct management of superficial thrombophlebitis/SVT
- Post vein surgery
- Post sclerotherapy
- Primary or secondary lymphedema in ISL stage II with no shape distortions
- Risk of rapid edema rebound
- Lipedema

4. Contraindications

JOBST® Confidence leg wear should not to be worn in cases of:

- Advanced arterial insufficiency including ischemia
- Uncontrolled congestive heart failure
- Untreated septic phlebitis
- Phlegmasia cerulea dolens
- Incompatibility to fabric

5. Cautions

- Weeping dermatosis
- Cutaneous infections
- Open wounds must be covered with an appropriate dressing before the compression garment is put on.
- Severely compromised skin sensibility and impaired sensitivity of the limb
- Advanced peripheral neuropathy
- Immobility (confinement to bed). Not recommended to wear during bed rest.
- Conditions in which increased venous and lymphatic return is not desired.
- Appropriate hygiene and skin care should be applied.
- Appropriate footwear must be worn when walking to prevent falls.
- Ill-fitting compression garments may cause constriction leading to e.g. skin necrosis, pressure ulcers and pressure related peripheral nerve damage.
- The attending physician decides whether compression in general can be applied to patients suffering from the following diseases: rheumatoid arthritis, complex regional pain syndrome (CRPS, M. Sudeck), malignant lymphedema, gangrene.
- Medical compression garments should only be used after consultation with your doctor or therapist and checked regularly by them on the appropriateness of your therapy, also considering age and health status.
- Patient measurements should be taken by a trained person to determine correct product fit.

- Patients should not wear a lower compression class in JOBST® confidence than indicated in the prescription or worn previously.
- Patients with pronounced skin folds and shape distortions should not wear JOBST® Confidence.
- Changing to a compression garment of different fabric may lead to variations in edema management.
- When re-ordering a garment after 6 months we recommend remeasuring the patient as the limb circumference might have changed.
- If you notice discomfort, pain, numbness, changes in skin color, appearance of new wounds or swelling during wear, stop using the product and contact your healthcare professional.
- Any serious incident that has occurred in relation to this device should be reported to the manufacturer and the competent authority of your state

6. Directions for storage / Guarantee

Made-to-measure products are products intended for immediate use and no expiry date for storage is indicated on these. Due to wear or slackening of the material, the medical efficacy can be guaranteed for a wearing period of six months. This assumes correct handling (e.g. product care, donning and doffing).

Please protect your compression garments from direct exposure to sunlight, heat, and moisture. We recommend storage at room temperature.

If the disease persists after 6 months, please consult your attending physician concerning the possible need for a new compression stocking.

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BSN-JOBST GmbH
Australian Distributor: BSN medical (Aust.) Pty Ltd